


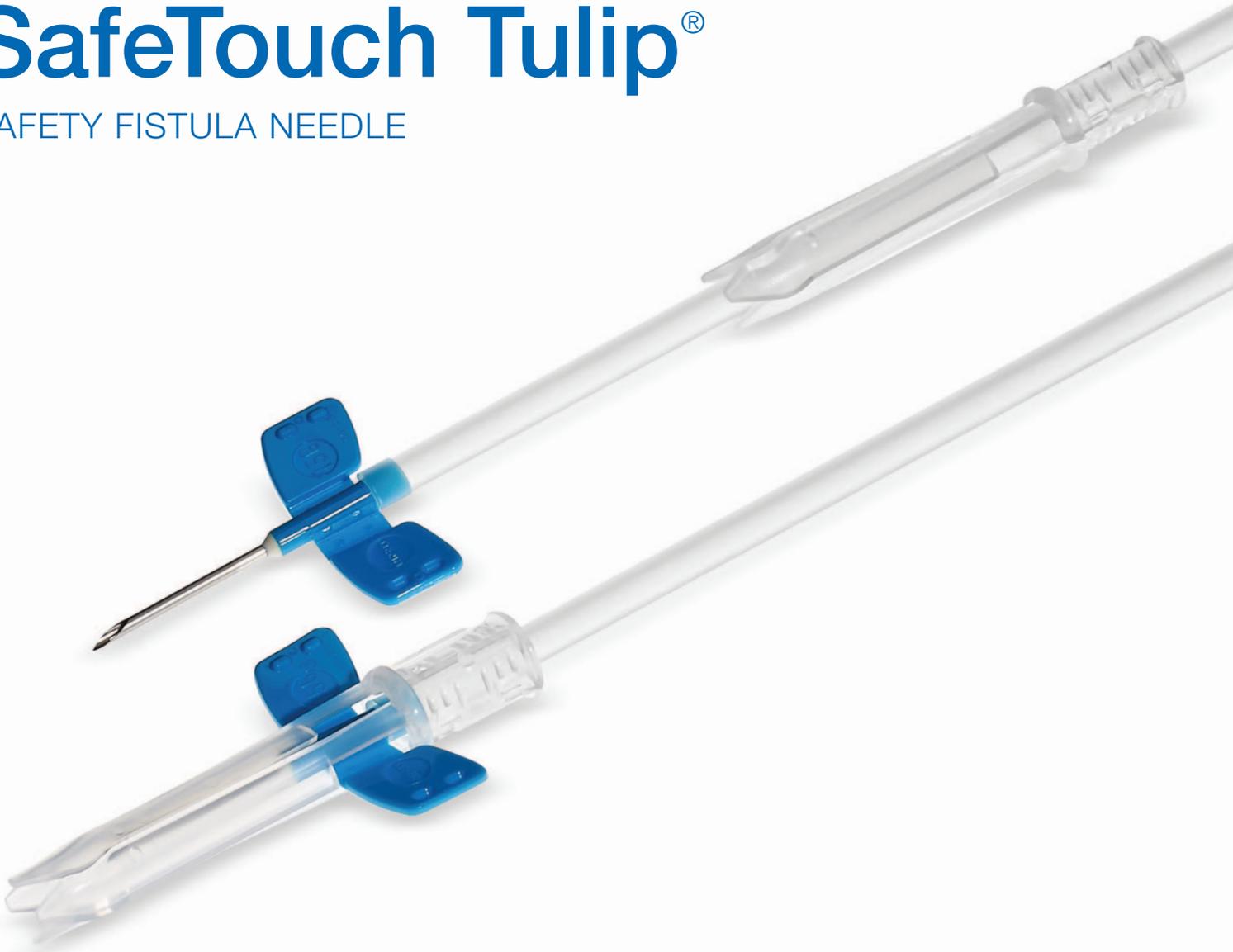

# SafeTouch Tulip<sup>®</sup>

SAFETY FISTULA NEEDLE



## Features and Benefits of the Tulip Safety Fistula Needle:

- Ease of use during patient cannulation and needle placement
- Audible click indicating safety guard is locked
- Ease of safety activation using a one or two-handed technique




## **NIPRO CONNECT<sup>®</sup>** **PROGRAM**

**Our commitment to strengthen every partnership.**

All Nipro renal products are backed by the Nipro Connect Educational Program – offering clinicians product evaluations, in-services, continuing education units (CEU's) and instructional materials.

# SafeTouch Tulip Needle Removal Technique

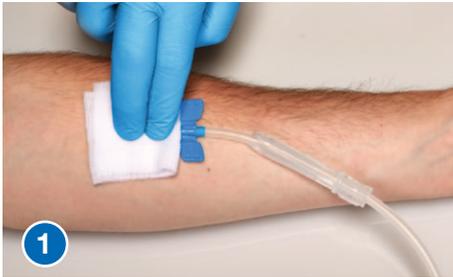

1

Remove all securing tape from the wings and the access area. Place a sterile dressing over the needle access site, and bring the safety guard forward towards the wing area.



2

While holding a sterile dressing over the needle access site, place a second finger at the end of the safety guard rigid holding area. Slide the Tulip safety guard forward until 2 petals of the Tulip safety guard tip are positioned over the flattened wings. Note, 2 different finger placement options are shown above.



3

While holding the safety guard firmly, grab and pull back on the tubing until the wings are locked, and the safety guard is fully secured, with an audible and tactile click.



4

Dispose of the SafeTouch Tulip needle in an approved biohazard sharps container.

## ORDERING INFORMATION

Item #	Description	Packaging
<b>FT+142530BC</b>	14GX1 Fixed Hub 12" Tubing	50/Box 6 Boxes/Case (300)
<b>FT+152530BC</b>	15GX1 Fixed Hub 12" Tubing	
<b>FT+153230BC</b>	15GX1.1/4 Fixed Hub 12" Tubing	
<b>FT+162530BC</b>	16GX1 Fixed Hub 12" Tubing	
<b>FT+163230BC</b>	16GX1.1/4 Fixed Hub 12" Tubing	
<b>FT+172530BC</b>	17GX1 Fixed Hub 12" Tubing	
<b>FT+173230BC</b>	17GX1.1/4 Fixed Hub 12" Tubing	
<b>FT+152530BCR</b>	15GX1 Rotating Hub 12" Tubing	
<b>FT+153230BCR</b>	15GX1.1/4 Rotating Hub 12" Tubing	
<b>FT+162530BCR</b>	16GX1 Rotating Hub 12" Tubing	
<b>FT+163230BCR</b>	16GX1.1/4 Rotating Hub 12" Tubing	
<b>FT+172530BCR</b>	17GX1 Rotating Hub 12" Tubing	

Item #	Description	Packaging
<b>FT+142530TP</b>	14GX1 Twin Pack 12" Tubing	2/Package: Red - Arterial Blue - Venous  120 Pairs + 10 Singles
<b>FT+143230TP</b>	14GX1.1/4 Twin Pack 12" Tubing	
<b>FT+152530TP</b>	15GX1 Twin Pack 12" Tubing	
<b>FT+153230TP</b>	15GX1.1/4 Twin Pack 12" Tubing	
<b>FT+162530TP</b>	16GX1 Twin Pack 12" Tubing	
<b>FT+163230TP</b>	16GX1.1/4 Twin Pack 12" Tubing	
<b>FT+172530TP</b>	17GX1 Twin Pack 12" Tubing	
<b>FT+173230TP</b>	17GX1.1/4 Twin Pack 12" Tubing	
<b>FT+142530TPR</b>	14GX1 Twin Pack Rotating Hub 12" Tubing	
<b>FT+152530TPR</b>	15GX1 Twin Pack Rotating Hub 12" Tubing	
<b>FT+162530TPR</b>	16GX1 Twin Pack Rotating Hub 12" Tubing	
<b>FT+172530TPR</b>	17GX1 Twin Pack Rotating Hub 12" Tubing	

**CAUTION:** Federal law restricts these devices to sale by or on the order of a physician.



**NIPRO MEDICAL CORPORATION**  
200 Crossing Blvd., Bridgewater, NJ 08807  
T: +1 (908) 393-7030, F: +1 (908) 393-7031  
[www.nipro.com](http://www.nipro.com)

© 2014 Nipro Medical Corp. All Rights Reserved.  
SafeTouch Tulip is a registered trademark of Nipro Corporation.  
Printed in U.S.A. RN-DSUS-TULIP Rev 0