41300700 / Rev. 01-25/01

4 Liter / 12 Seiten

Booklet, Zickzack-Falz, 23 Sprachen

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en: Instructions for use citric acid 50% - non-sterile
Intended use: Citric Acid 50% for heat disinfection, cleaning, and decalcification of haemodialysis monitor fluid pathway surfaces. **Usage and efficacy:** Citric Acid 50% at $\geq 80^{\circ}\text{C}$, concentration $\geq 20\text{ g/L}$ (2%) with an exposure time $\geq 15\text{ min}$, to guarantee bactericidal (EN 13727/14561 interfering substances high), fungicidal (EN 14562/13624 interfering substances high), virucidal (EN 14476 interfering substances low; EN 17111 interfering substances high) and tuberculocidal (EN 14348/14563 interfering substances high) effect, cleaning, and decalcification. Rinse fluid pathway after disinfection cycle with ISO 23500-3 compliant water at 700 mL/min for $\geq 8\text{ min}$. For exact haemodialysis monitor settings follow the instructions of the machine manufacturer. Citric Acid 50% is validated for the use on Nikkiso DBB 05/06/07, DBB-EXA; Fresenius 4008S, 5008S; Gambro AK 200S, AK 95S; and B. Braun Dialog EVO haemodialysis monitors. **Hazard statement:** Causes serious eye irritation. **Eye/face protection:** Wear safety glasses with side shields conforming to EN 166. **Skin/body protection:** Wear long-sleeved work clothing and protective gloves conforming to EU Directive 89/686/EEC and standard EN 374. Inspect gloves prior to use. Use proper glove removal technique to avoid skin contact with this product. Dispose of contaminated gloves after use in accordance with applicable laws and good laboratory practices. Wash and dry hands. After skin con-

MTN Neubrandenburg GmbH
 Gustav-Kirchhoff-Str. 2
 17033 Neubrandenburg, Germany
www.nipro-group.com

UK Responsible Person:
 Nipro Medical UK Ltd.
 Units 12-14 South Point,
 Ensign Way, Hamble,
 Hampshire
 SO31 4RF, United Kingdom

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tact, rinse off with plenty of water. **IF IN EYES:** Rinse continuously with water for several minutes. Remove contact lenses if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/attention. **Safety precautions:** Avoid contact with eyes and skin. Avoid formation of aerosols. Provide appropriate exhaust ventilation at places where aerosols are formed. Prolonged or repeated exposure may cause allergic reactions in certain sensitive individuals. Can be corrosive to metal. **Storage:** Store in a tightly closed original container in a well-ventilated area. Not compatible with strong bases and oxidants. Keep away from food, drink, and animal feed. Dispose of product residues in observance of waste directive 2008/98/EC and national and regional regulations. Empty contaminated packaging completely, it can be recycled. Disposal of packaging that is not completely emptied should be carried out in the same way as disposal of the product. **Warnings:** Do not use after the expiration date. Only use undamaged containers and clear colorless liquid. The volume of the canister is sufficient for several heat disinfection, cleaning, and decalcification cycles. The solution used for one cycle cannot be reused as the properties of the product cannot be assured. Close the container after the disinfection cycle. Once opened, the product must be used within 4 months. Any serious incident that has occurred in connection with the product must be reported to the manufacturer and the competent authority of the state in which the user is established.

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